Installation manual

MANUEL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONSANLEITUNG
MANUALE DI INSTALLAZIONE
MANUAL DE INSTALACIÓN
INSTRUKCJA INSTALACJI
PŘÍRUČKA PRO INSTALACI

Springer

Contents

Preface	4
1. This product	6
2. Scope of delivery	7
3. Assembly and adjustments	8
3.1. Tools	8
3.2. Seat depth	9
3.3. Backrest height	9
3.4. Width between armrests	10
3.5. Footrest depth (Seat frame adjustment)	11
3.6. Footrest length	12
3.7. Footrest width	12
3.8. Footplate position	13
3.9. Changing the battery	14
3.10. Changing wheels or tyres	15

Preface

To support you on the installation and repairs of this powered wheelchair, we provide you with this installation manual. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

The information in this manual applies to the following wheelchair(s):

- Springer

Important note

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	User manuals of applied operating console and battery charger For user and specialist dealer
	Installation instructions For specialist dealer
	Service manual for wheelchairs For specialist dealer
	Drawings of (spare) parts For specialist dealer
	EC declaration of conformity

1. This product

1. Backrest
2. Armrests
3. Arm pads
4. Operating console
5. Seat
6. Safety belt
7. Leg rests or footrests
8. Footplates
9. Steering wheels (front wheels)

10. Driving wheels (rear wheels)
11. Driving motors
12. Battery housing
13. Front light (optional)
14. Rear light (optional)
15. Anti-tipping
16. Identification plate

2. Scope of delivery

The following components are part of the delivery:

- Framework with armrests, operator control, rear and front wheels, seat and backrest
- Footrests
- Batteries
- Battery charger
- Tools
- User manual of:
 - wheelchair
 - Operating console
 - Battery charger
- Options

Before use, check if everything is included and that no products are damaged.

Please note that the basic configuration may differ in different European countries. Contact your specialist dealer.

3. Assembly and adjustments

CAUTION

Risk of injuries or damage

- Only use the settings described in this manual.
 - Only use parts described in this manual.
 - Only use genuine spare or replacement parts approved by Vermeiren.
 - Prevent bystanders from accessing the working area.
 - Keep the (dis)assembling area clean and tidy, to prevent falling.
 - Collect all small parts in a box or bag.
 - While (dis)assembling and operating the wheelchair, make sure no objects or body parts get caught between moving parts. In particular, pay attention to the position of the cables.
 - Turn the wheelchair off before you perform any mechanical adjustments and make sure that the free-wheel lever is in braking position.
 - Make sure all screws are firmly secured before driving.
 - Software changes are only allowed if executed by skilled personnel, taking into account the user and environment (stability, acceleration, indoor/outdoor, etc.).
-  Read the user manual of this wheelchair. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.


3.1. Tools

To (dis)assemble and adjust the wheelchair, the following tools are needed:

- Wrench no. 10
- Allen key no. 4, 5
- Screwdriver


3.2. Seat depth

1. Loosen the two M6 socket screws (25) on either side of the frame.
2. Slide seat frame (26) backwards or forwards to get the desired position.
Make sure that inner and outer tube holes are in line.
3. Insert the 4 socket screws (25) and tighten the nuts firmly.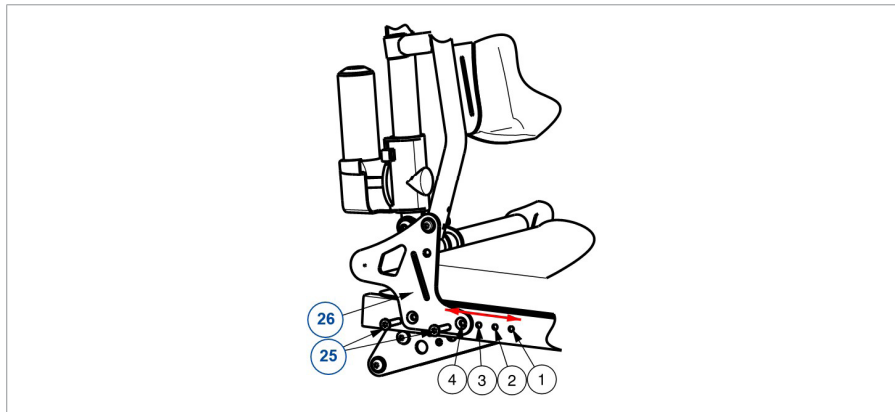


3.3. Backrest height

The backrest height can be adjusted in 2 ways: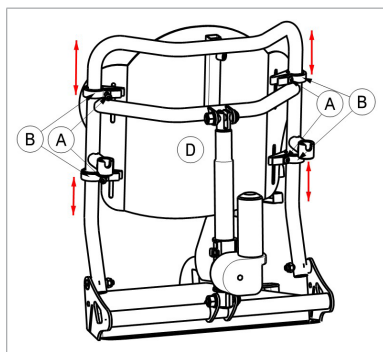

3.3.1. Method 1

1. Loosen the screws (A) on the backrest clips (B) a little.
2. The backrest clips (B) come open.
3. Move the backrest clips (B) and position the backrest plate (D) in the desired height (range of 50 mm).
4. Retighten the screws.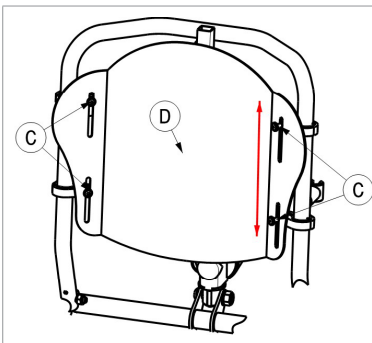


3.3.2. Method 2

1. Loosen the screws (C) on the backrest plate.
2. Move the backrest plate (D) to the desired backrest height (range of 60mm).
3. Retighten the screws.

3.4. Width between armrests


To adjust the width of the armrests:

1. Loosen the screw (A) on the underside of the seat frame.
2. Adjust the armrest to a comfortable width.
3. Retighten the screw (A).
4. Repeat this at the other side of the wheelchair.
5. Make sure the armrests are positioned identically on both sides of the seat.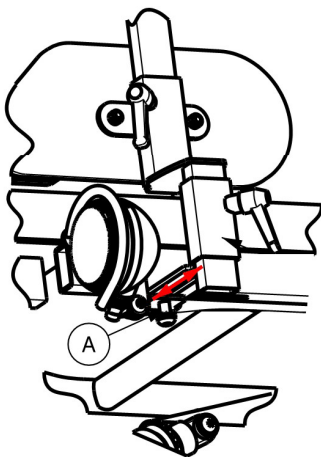

3.5. Footrest depth (Seat frame adjustment)

1. Remove the screws (A) on the side of the seat frame.
2. Slide the footrest forward/backward to the desired position (range: 30mm).
3. Retighten the screws.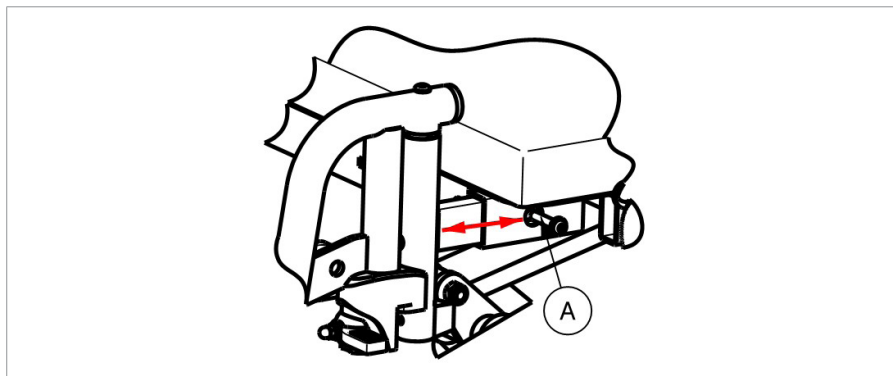


3.6. Footrest length

⚠ CAUTION

Risk of damage


- Make sure that the footrests won't touch the ground. Keep a minimum safe distance of 60 mm above the ground.
- Do not adjust the footrest to a position where the footplate touches the outer tube.

Adjust the length of the footrest as follows (6 possible positions):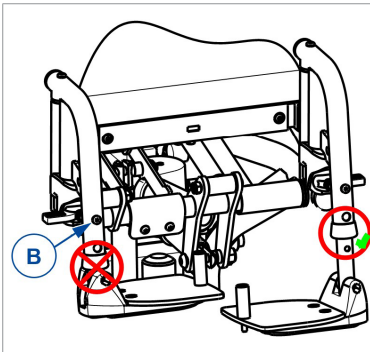


1. Remove the screw (B).
2. Adjust the footrest to a comfortable length.
3. Retighten the socket screw firmly.
4. Repeat this for the second footrest. Make sure the footrests are positioned identically.

3.7. Footrest width

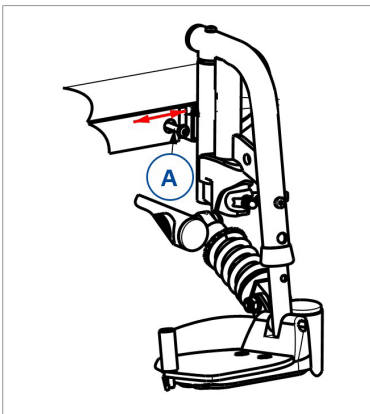
1. Loosen the screw (A) on the front of the seat frame.
2. Move the footrest in a horizontal direction until the desired position.
3. Retighten the screws (A) properly.
4. Repeat instructions above for the other footrest.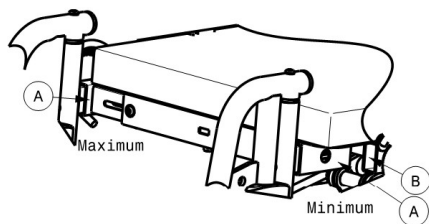

The maximum position of the footrest is when the square tube of the footrest (A) has the same position as the outside square tube of the seat frame (B). Otherwise you will damage the front lights.

3.8. Footplate position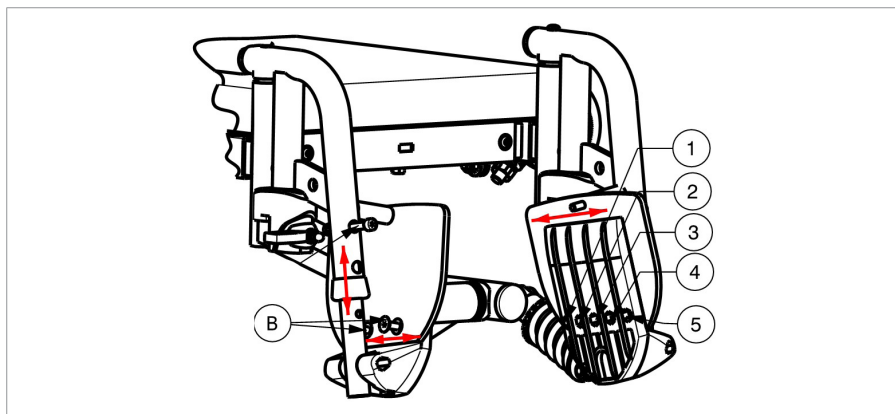

Adjust the depth of the footplate as follows (3 positions):

1. Remove the screws (B).
2. Adjust the depth of the footplate to a comfortable position.

Position 1	Hole 1 and 3
Position 2	Hole 2 and 4
Position 3	Hole 3 and 5

3. Retighten the screws.

3.9. Changing the battery



CAUTION

Risk of injuries or damage

- The batteries must only be replaced by trained personnel.
- Do not come into contact with the acid from the batteries.
- Replace the batteries in a well-ventilated room.
- Make sure that the ventilation holes of the battery housing are free from any contaminations.
- First read the charging instructions in the user manual before charging the batteries.
- Do not place fingers between the batteries and the battery housing.

 Replace both batteries at the same time and with the same type as the ones used in the wheelchair. If this is not possible, contact the nearest Vermeiren facility.

1. Switch OFF the wheelchair.
2. Remove the battery cover (1) from the hook-and-loop connections.
3. Disconnect all the wires on the batteries.
4. Remove the power module (2).
5. Lift the batteries out of the battery housing.
6. Take the new batteries.
7. Place the batteries in the battery housing.
8. Connect all the wires to the batteries.
9. Replace the battery cover by attaching it to the hook-and-loop connections.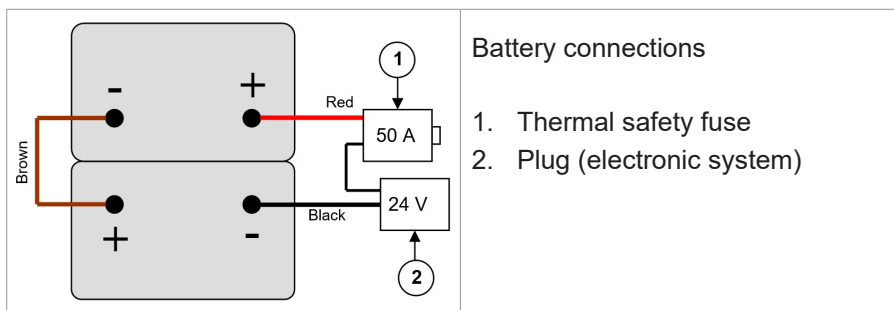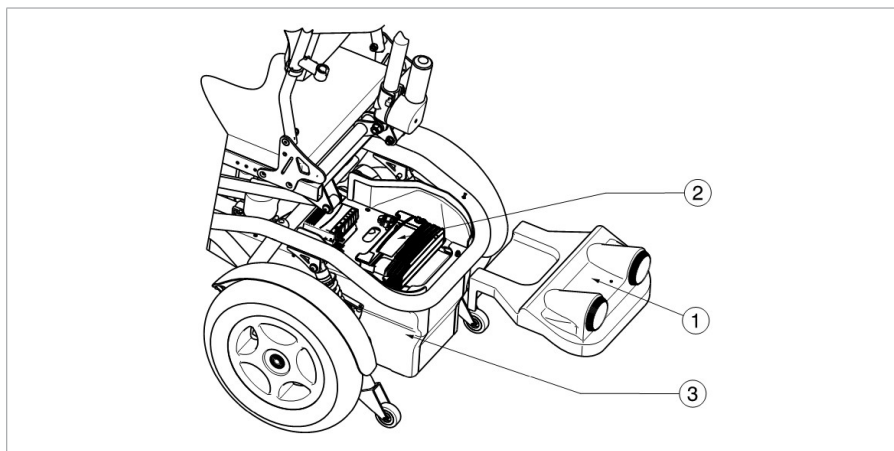

3.10. Changing wheels or tyres

⚠ CAUTION


Risk of injuries or damage

- Make sure that all air is released from the tyre, before removing it.
- Use a wooden block and jack to lift the wheelchair. Always place the jack under the lower frame, never under any other parts.
- Make sure that no body parts, the inner tube nor any other objects are pinched between tyre and rim.
- Handle carefully to prevent damage to the rim.

1. Replace the inner tube, and if necessary also the tyre.
2. Inflate the tyre using inflating equipment which complies to regulations and with a pressure indication in “bar”.
3. Make sure that the pressure is correct, see the pressure indication on the tyres.

3.10.1. Driving wheels

1. Lift the driving wheel from the ground using a jack.
2. Remove the socket screws (24) and take the wheel from the flange.
3. Open tyre valve and push it gently to release the air.
4. Loosen the 5 screws that connect both rim halves (on the inner rim half).
5. Separate the rim halves and remove the inner tube from the tyre.
6. Inflate the new or repaired inner tube partly.
7. Insert the inner tube into the tyre.
8. Place the two rim halves back on the tyre. Make sure that the tube valve pops out of the opening in the rim.
9. Connect the rim halves with the socket screws. Tighten by hand.
10. Inflate the tyre at the correct pressure, see the indication on the tyre.
11. Remove grease and other contamination from the thread of the 5 flange socket screws. Apply screw adhesive (ex. Loctite) to these screws.
12. Mount the wheel on the flange.
13. Lower the jack.
14. Make sure the socket bolts are firmly secured.

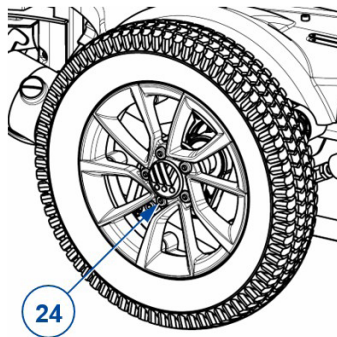3.10.2. Steering wheels

3.10.2.1. Dismantling

1. Loosen the screw on the steering wheel axle (25) and remove it from the steering wheel fork.
2. Let the air out of the steering wheel by lightly pressing the pressure pin on the valve.
3. Loosen the 5 screws that hold the split rim together (26).
4. Now separate the rim sides.

3.10.2.2. Assembly

1. Insert the partly-filled inner tube into the tyre.
2. Connect the two sides of the rim through the tyres and use the 5 connecting screws to screw the rim together (26).
3. Make sure that the valve juts out of the valve opening in the rim.
4. Put the wheel back into the front wheel fork (25) and inflate the wheel.

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All rights reserved, including translation.