VERMEIREN

Paris

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN

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EN Contents

1.	Technical details	2
2.	Components	2
3.	Explanation of symbols	2
4.	Check after delivery	2
5.	Intended use	2
6.	Assembly / Dismantling	3
7.	Use of the swivel seat	3
8.	Carrying the swivel seat	4
9.	Safety instructions	4
10.	Care	4
11.	Disinfection	5
12.	Repairs / Service / Inspection	5
13.	Shipping and storage	5
14.	Warranty	5
15.	Disposal	5
16.	Declaration of conformity	5

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INSTRUCTION MANUAL
Swivel bath transfer seat "Paris"

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout
Type	Swivel bath transfer seat
Model	Paris
Width	515 mm
Length	721 mm
Height	560 mm
Dismantled height (backrest, arm supports removed)	235 mm
Seat width	410 mm
Seat depth	370 mm
Backrest height	325 mm
Inner width bathtub	560 - 660 mm
Weight total	6,65 kg
Weight frame	5,55 kg
Weight arm supports	0,50 kg
Weight backrest:	0,65 kg
Patient weight	max. 130 kg
Storage temperature	+5°C - +41°C
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg	

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the seat, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your seat. Please remember that observance of our hints would keep your product in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

2. Components

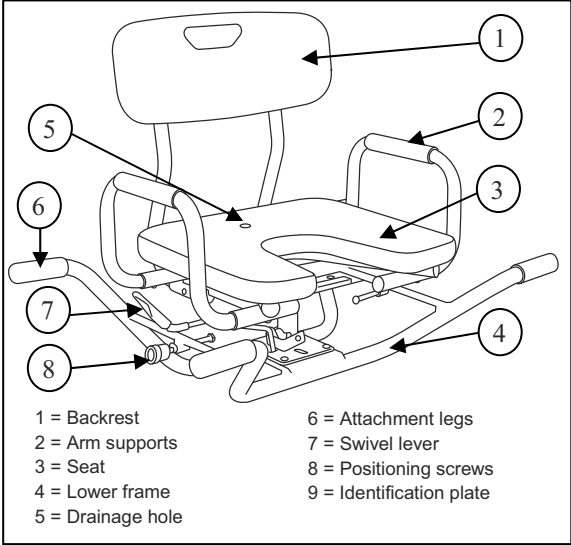
Fig 1. Components Swivel seat

3. Explanation of symbols

- Maximum mass
- Indoor use
- CE conformity
- Read the instruction manual before use
- Observe the safety instructions

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- Swivel bath transfer seat
- Arm supports
- Backrest
- Manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

The seat shall be used by 1 person.
The seat shall be used indoor, in a bathtub

Is your bathtub too small for a conventional tub / shower safety / transfer seat? Then the swivel bath transfer seat is the solution. The seat is ideal for people who feel insecure to sit on a bath board. The swivel seat can be used for a bathtub with inner width of 560 until 660 mm.

The swivel bath transfer seat is a 360° turning bath seat that allows easy transfer over the bathboard. The seat can be locked into four 90 degree positions. The user sits on the seat from the side of the bath and then swivels over the bath for a wash or shower.

The construction of the seat allows full use by elderly people or during revalidation in home environment.

The seat shall not be used as toy for children. The seat should only be used on horizontal, non smooth surfaces of the bathtub board. The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

6. Assembly/Dismantling

- ⚠ **CAUTION: Chance of pinching - Do not place fingers between the components of the seat.**
- ⚠ **CAUTION: Risk of clamping - Keep fingers, cloths, jewelry or items away from moving parts of the seat.**

The Paris seat is foreseen with removable arm supports, removable backrest and has a swivel seat that can be locked in four 90° positions.

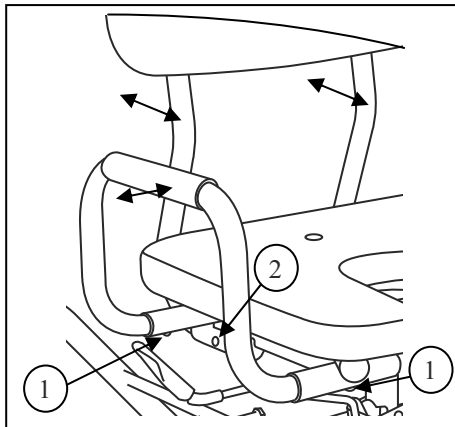
Fig 2. Mounting / removing arm supports and backrest

6.1. Removing arm supports

1. Push on the two pens ① located on the bottom side of the arm support tubes (just below the seat).
2. Pull on the arm support.

6.2. Mounting arm supports

1. Push the tubes of the arm supports in the tubes of the lower frame.
2. Check that the arm supports are well fixated. When you hear that the pens ① clicks in the holes, the arm supports are good fixated.

6.3. Removing backrest

1. Push on the two pens ② located on the outside of the backrest tubes (just below the backside of the seat).
2. Pull the backrest backwards.

6.4. Mounting backrest

1. Push the tubes of the backrest in the tubes of the lower frame.
2. Check that the backrest is well fixated. When you hear that the pens ② clicks in the holes the backrest is well fixated.

7. Use of the swivel seat

The swivel bath transfer seat is ideal for usage in a bathtub. The specially designed rubbers at the end of each attachment leg will create a safe stability for the seat and avoid the sliding over the bath board.

The seat can be mounted on the top of the bathtub and can be easily removed when not in use. So other persons can use the bath.

7.1. Mounting the swivel seat to the bathtub:

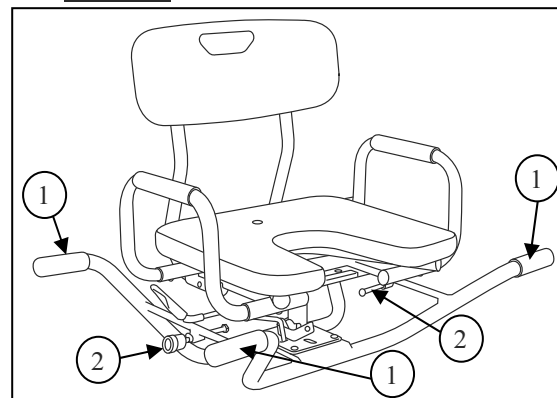
Fig 3. Mounting / removing swivel seat to the bath

- ⚠ **CAUTION: Risk of injury - Check that the seat is properly fixed in the bathtub with the positioning screws.**
- ⚠ **CAUTION: Risk of injury - Check that at least 30 mm of each attachment leg rest securely on each side of the bath.**

1. Place the swivel seat with the four attachment legs ① on the board of the bathtub.
2. Turn the positioning screws ② on both sides against the bath.
3. Turn the wing screw against the lower frame.
4. Check that the swivel chair is properly fixated to the bath.

7.2. Removing the swivel seat from the bathtub:

1. Loosen the wing screw.
2. Loosen the positioning screws ② from the bath.
3. Take the swivel seat out the bath.

7.3. Turning the swivel seat:

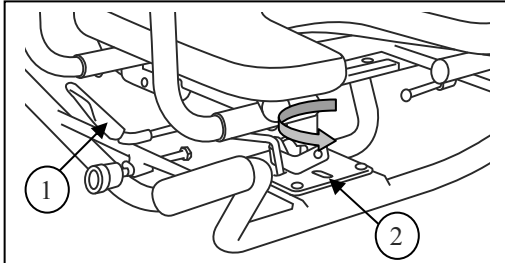
Fig 4. Turning the swivel seat

⚠ CAUTION: Risk of injury - Check that the seat is properly locked after turning.

Let someone assist you for a smoothly, safely turning of the swivel bath transfer seat.

1. Pull up swivel lever ①. The lever comes out the locking hole ②.
2. Lift up the legs of the patient when turning to the washing position.
3. Turn the seat to the desired position. The seat can be locked in four 90° positions by placing the lever in one of the four locking holes ②.
Desired positions:
 - **Patient position for washing:** place seat parallel with lower frame
 - **Patient position for transfer to/from swivel seat:** place seat perpendicular with lower frame
4. Lock the seat by placing the swivel lever ① in the correct locking hole ②.
5. Check that the seat is properly locked.

7.4. Transfer to/from swivel seat:

⚠ CAUTION: Risk of injury - Be careful that you not slip out on the bathroom floor.

For people who can walk:

1. Walk to the swivel seat.
2. Turn the swivel seat in a good position, it must be easy to transfer to the seat. (seat perpendicular with the lower frame)
3. Place yourself in the seat. For extra support you can grasp the arm supports. **Be careful that you not slip out on the floor.**
4. Turn the swivel seat to the washing position.

For persons who has not the walking ability:

1. Let someone assist you to transfer to / from the swivel seat.
2. Turn the seat in the good position. (seat perpendicular with the lower frame).
3. Lift the patient in the swivel seat. For extra support the patient can grasp the arm supports.
4. Turn the swivel seat to the washing position.

When the swivel seat is placed too high for performing the transfer on a safe manner, you can make use of a step stool/bench.

8. Carrying the swivel seat

The best way to carrying the swivel seat is to grasp the seat on the handgrip of the backrest.

9. Safety instructions

- **Do not exceed the maximum user weight. The maximum user weight for your seat is 130 kg.**
- Use the swivel seat on horizontal, non-smooth surfaces of the bathtub board.
- Keep it away from children. The swivel seat shall not be used as a toy.
- When it is used by, on or near children, invalids or disabled persons close supervision is necessary.
- Risk of burns – Be careful when using in extreme hot or cold environments for a sufficient amount of time and when touching.
- Do not place fingers between components of the swivel seat.
- Keep fingers, cloths, jewelry or items away from moving parts of the swivel seat.
- Check that the seat is properly locked after turning.
- Check that the seat is properly fixed in the bathtub with the positioning screws.
- Check that at least 30 mm of each attachment leg rest securely on each side of the bath.
- Do not use the swivel seat when it is not working properly.
- Only Vermeiren approved accessories are allowed.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

10. Care

It is recommended to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish consult your specialist dealer.

Regular care would ensure that your swivel seat is preserved in a perfectly functional condition.

Clean plastic parts of your swivel seat with commercial plastic cleaners. Only use a soft brush or soft sponge. Examples of plastic parts are the seat and backrest.

11. Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and plastic surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: www.rki.de). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

12. Repairs / Service / Inspection

Lifetime of the swivel seat is influenced by its use, storage, regular maintenance, servicing and cleaning.

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the seat, it should be serviced according to the following criteria:

- Completeness
- Working of swivel mechanism
- Positioning screws
- Frames (deformation, stability, connections)
- Cleaning
- **Damaged varnish (remove possible rust and repaint)**

Every 8 weeks:

- Lubricating (moving parts shall be greased)

Every 6 months or for every new user, it should be serviced according to the following criteria:

- General review

- Disinfection

13. Shipping and storage

The shipping and storage of the swivel bath transfer seat shall be according following instructions:

- Store in a dry place (between +5°C and +41°C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the product from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- The product must be stored without being subjected to strains. (Do not put too heavy parts on the product, not clamping between something, ...)

14. Warranty

Excerpt from the "General Business Conditions".

(...)

5. The guarantee period for warranty claims is 24 months.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

15. Disposal

When disposing of the swivel seat, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

16. Declaration of conformity

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V.

Address :

Vermeirenplein 1/15

2920 Kalmthout

Belgium

declares under his sole responsibility that the CE marked medical devices :

Productgroup: Aids: Swivel bath transfer seat

Brand: Vermeiren

Type: Paris

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,

and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC: 2007

and is in conformity with the relevant European harmonized standards:

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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N.V. VERMEIREN N.V.
Vermeirenplein 1/15
B-2920 Kalmthout
BELGIUM

Notes

A series of horizontal dotted lines for taking notes.

Notes

A series of horizontal dotted lines for taking notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Reatime S.R.L.

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.reatime.it
e-mail: info@reatime.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Spain

Vermeiren Iberica, S.L.

Trens Petits, 6. - Pol. Ind. Mas Xirgu.
17005 Girona
Tel: +34 902 48 72 72
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

L. Vermeiren Ges. mbH

Winetzhammerstraße 10
A-4030 Linz
Tel: +43(0)732 37 13 66
Fax: +43(0)732 37 13 69
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Hühnerhubelstraße 59
CH-3123 Belp
Tel: +41(0)31 818 40 95
Fax: +41(0)31 818 40 98
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

The Netherlands

Vermeiren Nederland B.V.

Domstraat 50
NL-3864 PR Nijkerkerveen
Tel: +31(0)33 2536424
Fax: +31(0)33 2536517
website: www.vermeiren.com
e-mail: info@vermeiren.be

Czech Republic

Vermeiren ČR S.R.O.

Sezemická 2757/2 - VGP Park
193 00 Praha 9 - Horní Počernice
Tel: +420 731 653 639
Fax: +420 596 121 976
website: www.vermeiren.cz
e-mail: info@vermeiren.cz

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All rights reserved, including translation.