


Bill of Material			
No.	Plan	Article	Quantity
1	YP06972	2743765	2
2	P00050248	2670465	2
3	XP02231	2993879	2
4	XP02229(R)	2993877	1
	XP02230(L)	2993878	1
5	XP02079	GP15300	4
6	P00107339	GP32226	4
7	XP01914(L)	GP13300	1
	XP01915(R)	GP13400	1
8	XP02922	GP22126	4
9	---	1007521	2
10	M3	2994899	4
11	M6x20 Black	2994010	4
12	M6 DIN 125	2994414	4
13	XP03430	2997530	2
14	M6 DIN 985 Black	2994063	4
15	MBL	2990321	2
16	YP09506	26610...	2
17	M4 DIN985 Black	1907202	4
18	M4 DIN125 Black	1003303	4
19	XP03432	2997532	4
20	XP03431	2997531	4
**	P00093408	2998564	4
21	XP00412	2990456	8
22	M4x45 DIN912	2998457	4
23*	P00071746	26852...	2
24	YP02362(R)	18987...	1
	YP02363(L)	18986...	1
25	RIVET 4x14	1001584	4

Quantity per left and right side

*Pos. 23 for 22 inch wheels

** Use only with tube armrest.

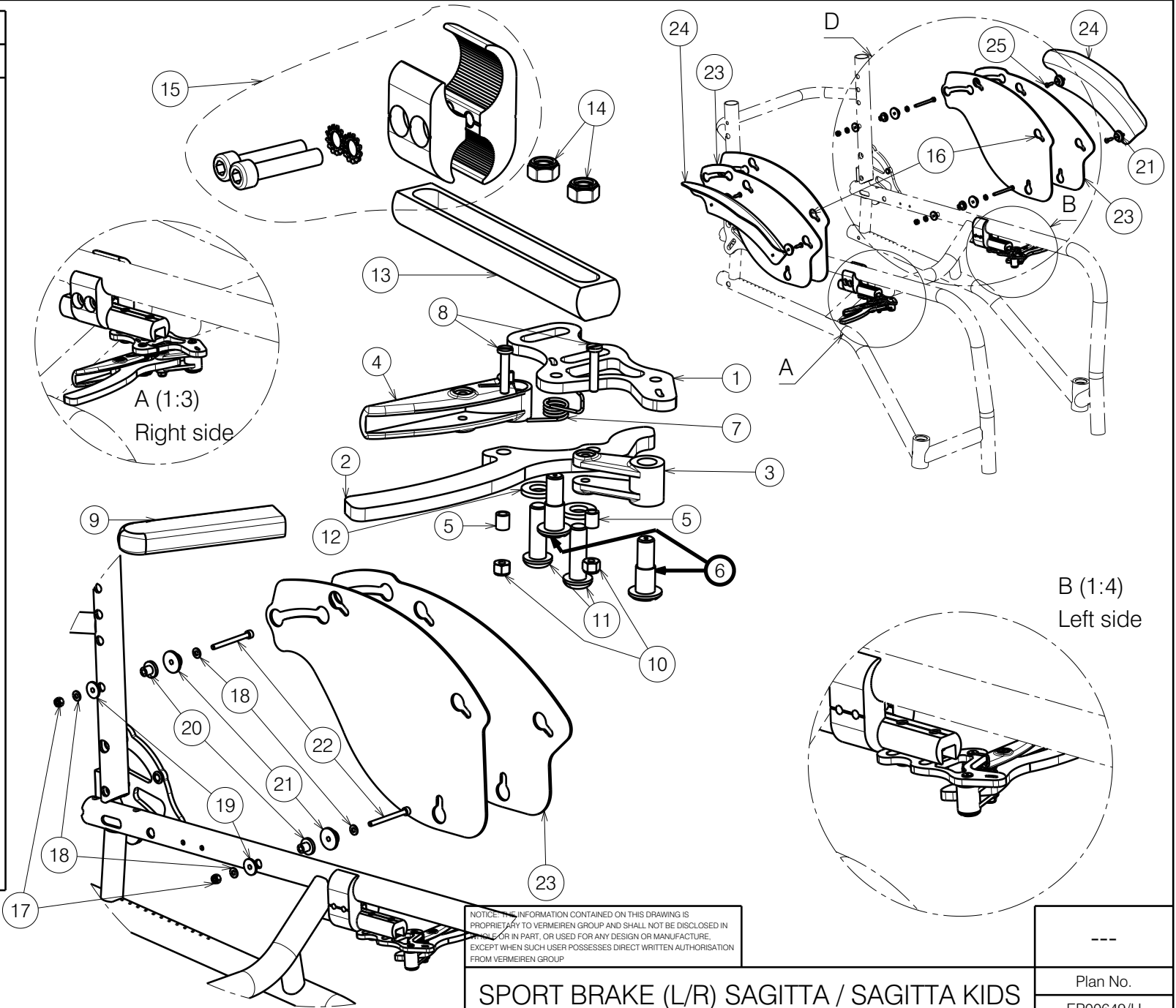

Detail D
Scale: 1:5

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

SPORT BRAKE (L/R) SAGITTA / SAGITTA KIDS

Scale:	1:2	Drafted	Verified	Confirmed
Date	15/11/2022	12/03/2026	12/03/2026	
Initials	WM	PM	AS	

Plan No.

EP00649/H

