



Electrical components presented in 2992525

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP



Sheet 1/3

<p>TURIOS ELECTRICE SEAT SE40 BACKREST SE36</p>				---
				Plan No.
				EP00624/1.1
Scale:	1:7	Drafted	Verified	Confirmed
Date	09/11/2021	2025-11-27	2025-11-27	
Initials	MK	FD	MK	



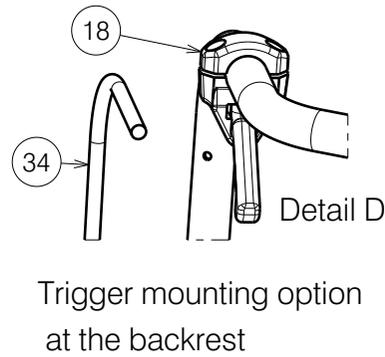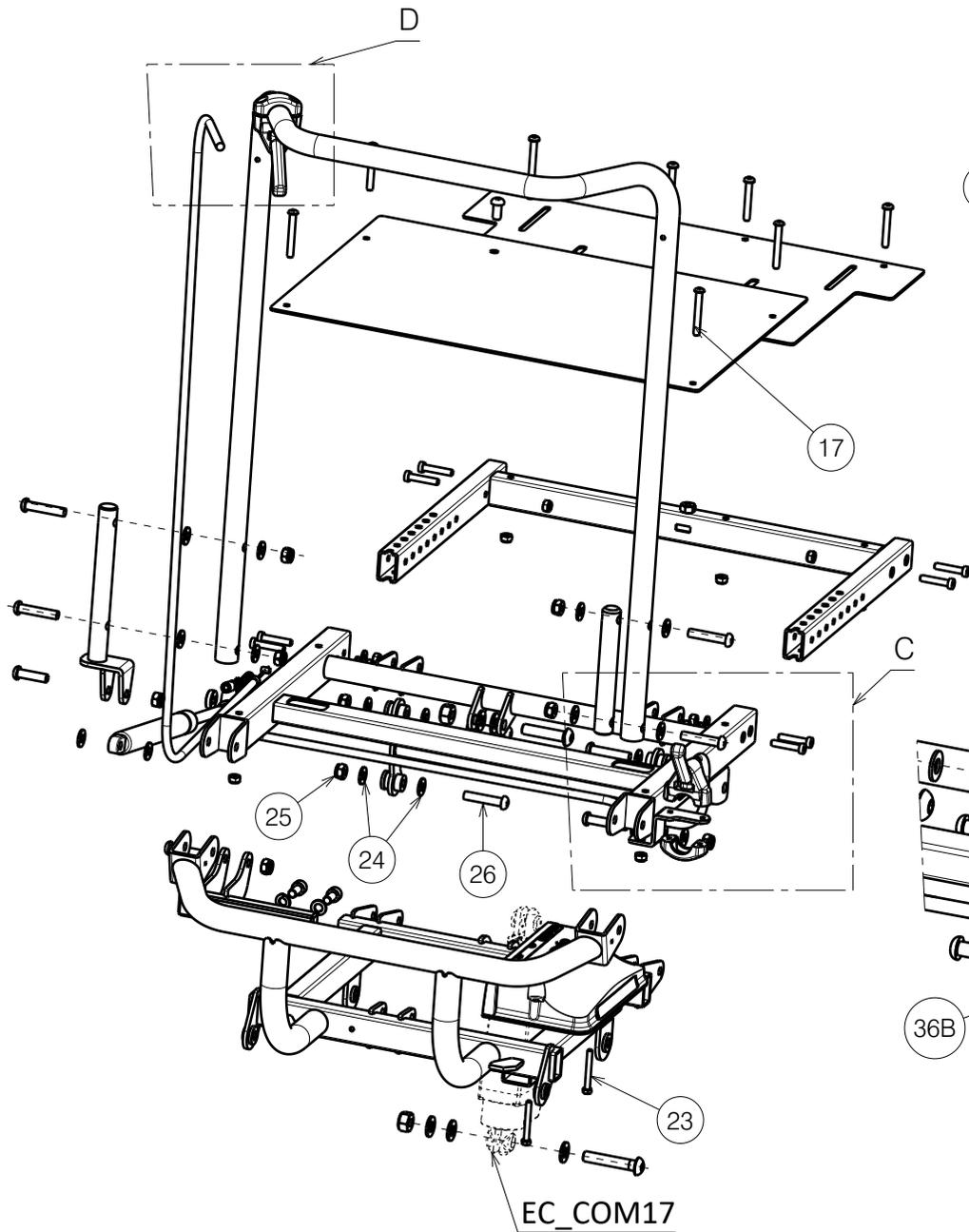

13 (s) and 14 (s) are only for small size (39cm seat width)

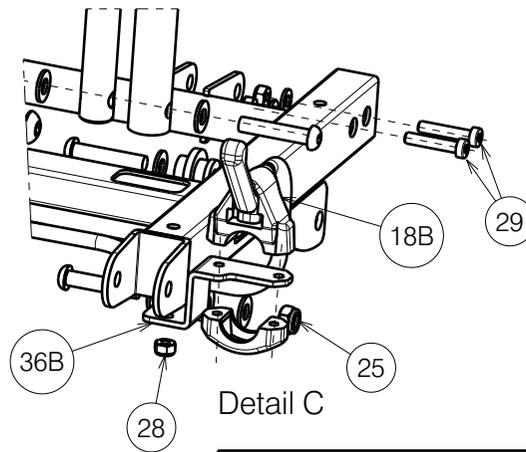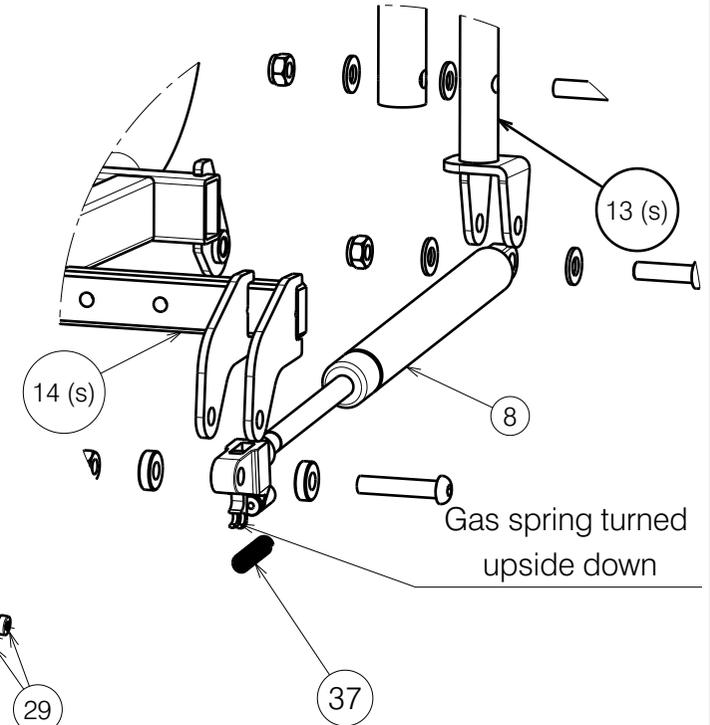

Trigger mounting option at the seat

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 2/3

TURIOS ELECTRICE SEAT SE40 BACKREST
SE36

Plan No.

EP00624/1.1

Scale: 1:7	Drafted	Verified	Confirmed
Date	09/11/2021	2025-11-27	2025-11-27
Initials	MK	FD	MK



Bill of Material				Bill of Material				Bill of Material			
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	YP07411 (39)	2776954	1	8	GAS SPRING 275N	1991628	1	29	M6X30BLACK	1915416	8
	P00080268(42)	2689854	1	9	-	-	-	30	M6 DIN125	1001564	10
	YP07412 (45)	2777054	1	10	-	-	-	31	M8x16	1991858	2
	P00080270(48)	2689954	1	11	TRIGGER	1994748	1	32	M6x40	1006111	4
2	YP07413 (51)	2777154	1	12	X004967	1908262	2	33	XP02856	GP21400	1
	YP07414 (39)	2777254	1	13	P00031739 (39,42)	26483	1	34A	Cable hose	1003279	0,96m
	P00080198(42)	2688954	1		YP09272 (45,48,51)	26428	1	34B	Cable hose	1003279	0,70m
	YP07415 (45)	2777354	1	14	P00028106 (39,42)	26482	1	35A	Cable line	1992149	0,96m
3C	P00080218(48)	2689054	1		YP09267 (45)	26427	1	35B	Cable line	1992149	0,70m
	YP07416 (51)	2777454	1		P00031741 (48,51)	26484	1	36B	YP09313	2643754	1
	YP07417 (39)	2777554	1	15	M8x16	1915411	2	37	Spring	2992121	1
	P00080264(42)	2689154	1	16	Washer M8	1001570	2	Material SE30			
3D*	YP07418 (45)	2777654	1	17	M8x30	1992717	2	No.	Plan	Article	Quantity
	P00080267(48)	2689254	1	18	Brake Handle	1996806	1	19	M10x55 ISO 7380	1998518	1
	YP07419 (51)	2777754	1	19	M10x55 ISO 7380	1998518	1	20	M10 DIN125	1001558	5
	YP08368 (39)	2614954	1	20	M10 DIN125	1001558	9	21	M10 DIN985	1001683	3
4	P00080260(42)	2689354	1	21	M10 DIN985	1001683	4	22	M10x45	1908848	1
	YP08369 (45)	2615054	1	22	M10x45	1908848	3	26	M8x40	1008739	1
	P00080262(48)	2689454	1	23	M5x40 DIN933	1001427	2	38	YP07736	2781754	1
	YP08370 (51)	2615154	1	24	M8 DIN125 BLACK	2994415	16	39	YP09294	2643354	1
5	YP07421	2777954	1	25	M8 DIN985	1001682	10	Electrical components presented in 2992525			
	YP07682 (39)	2600854	1	26	M8x40	1008739	8				
	P00080272(42)	2690454	1	27	M6x45	1991017	4				
	YP07683 (45)	2600954	1	28	M6 DIN985	2994063	15				
6	P00080293(48)	2690554	1	* - Use with BZ7/8 and BZE A - Standard mounting at the backrest SE36B B - Trigger mounting option at the seat SE36S							
	YP07684 (51)	2601054	1								
	YP07706 (39)	2779754	1								
	P00080301(42)	2690254	1								
7	YP07707 (45)	2779854	1	NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.							
	P00080303(48)	2690354	1								
	YP07708 (51)	2779954	1								
	YP07709 (39)	2780054	1								
7	P00080319(42)	2690054	1	Sheet 3/3							
	YP07710 (45)	2780154	1								
	P00080302(48)	2690154	1								
	YP07711 (51)	2780254	1								

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 3/3

TURIOS ELECTRICE SEAT SE40 BACKREST				Plan No.
SE36				EP00624/1.1
Scale: 1:7	Drafted	Verified	Confirmed	
Date	09/11/2021	2025-11-27	2025-11-27	
Initials	MK	FD	MK	