
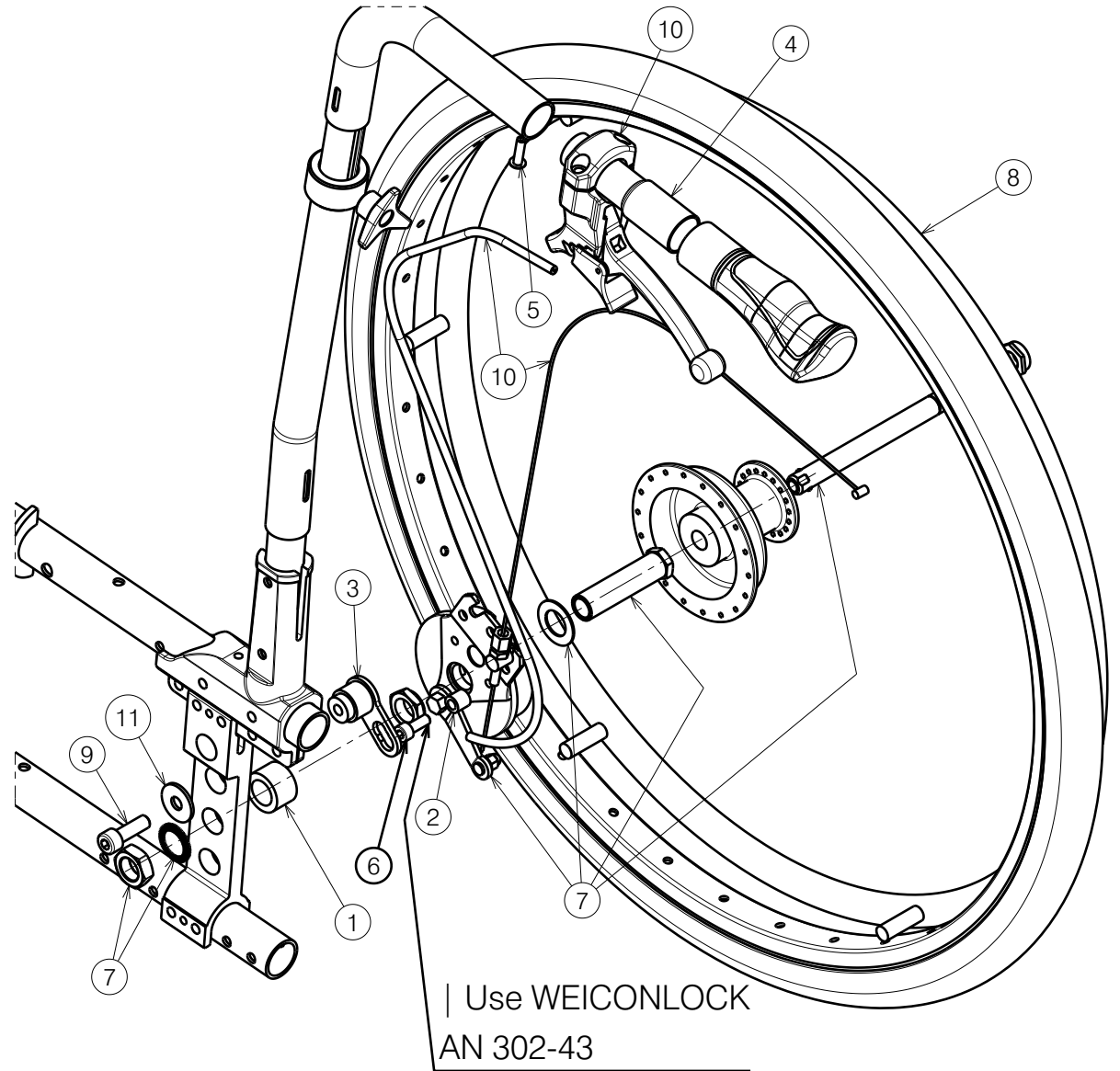


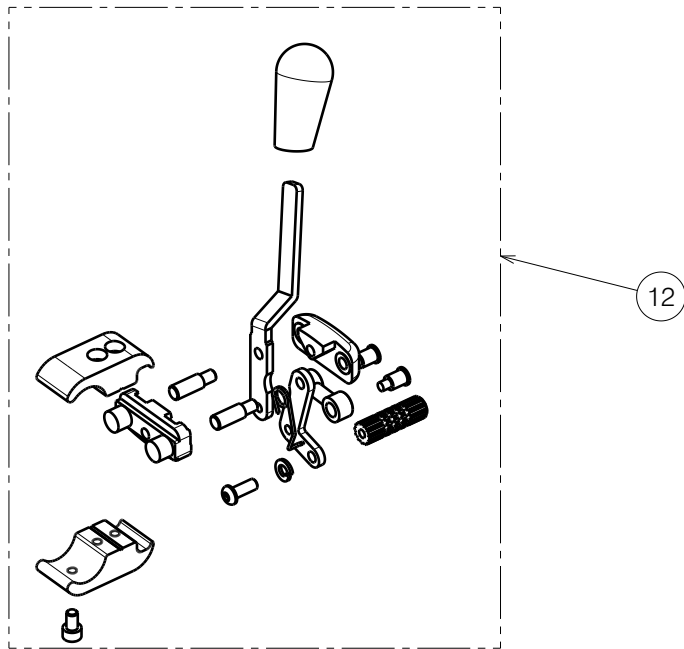

Material			
No.	Plan	Article	Qty
1	X003986	1902199	2
2	X005545	2019900	2
3	Y016096	1737800	2
4	YC00662	FC00428	2
5	D5x18	1001589	2
6	M6x16 DIN912	1001492	2
7	Left set	3990705	1
	Right set	3990708	1
8	24" Black PU	1914616	2
9	M8x25 DIN912	2994372	2
10	D25.4	3990707	2
11	M8 DIN9021	2994911	2
12	E00023606	1918195 (L)	1
	E00023606	1918196 (R)	1

Quantities are for one set
Right side depicted only

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

V500LD DRUM BRAKE WHEEL B74/B80

Scale:	1:4	Drafted	Verified	Confirmed
Date	5/09/2024	21/04/2026	21/04/2026	
Initials	SD	AZ	AS	

1921720

Plan No.
E00088414/C1.1

